DHEA-Sulfate kit

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2. INTENDED USE

DHEA-Sulfate kit is provided for the **quantitative** determination of **dehydroepiandrosterone sulfate** in human serum.

This test has 2 complete sets:

REF 20-01 for manual use;

REF 20-01 A must be used with ELISA automatic instrument “Alisei” manufactured by NEXT Level S.r.l. hereinafter referred to as analyser “Alisei”.

Instructions for use are described in:

section 13 for manual test,

section 14 for analyser

Note 1: *Take into account that calibrators’ nominals can be different for manual and automatic test kits.*

Note 2: *We guarantee applications of the test*

REF 20-01 A only on analyser “Alisei”,

REF 20-01 only for manual use.

While using a non-predefined method of use it is under end user responsibility to make sure that it was appropriately tested for ELISA kits.

Dehydroepiandrosterone sulfate (DHEA-S) is a steroid hormone produced mainly by adrenal cortex. The function of DHEA-S in the organism is not definitely specified. The hormone has a weak androgenic effect. In peripheral tissues it can be converted into testosterone – a very strong androgen. DHEA-S does not bind to special transport proteins in blood stream.

There is no circadian rhythm in DHEA-S secretion.

Quantitative determination of DHEA-S is necessary for differential diagnostic of ovariothy as well as for evaluation of adrenal function in pubescence and for differential diagnostics of Cushing syndrome and disease.

3. PRINCIPLE OF THE TEST

The **DHEA-Sulfate kit** is a solid phase immunoassay, based on competition between horseradish peroxidase-labeled DHEA-S and DHEA-S of serum sample for polyclonal anti-DHEA-S antibodies. After incubation on shaker enzyme-labeled DHEA-S not bound at the solid phase is removed by repeated washing. Quantity of the bound conjugate is inversely proportional to DHEA-S level in the sample.

During the incubation with TMB solution the colour is developing. The intensity of the colour is inversely proportional to DHEA-S concentration in specimens. DHEA-S concentration in the patient sample is read from a standard curve that is processed in each assay.

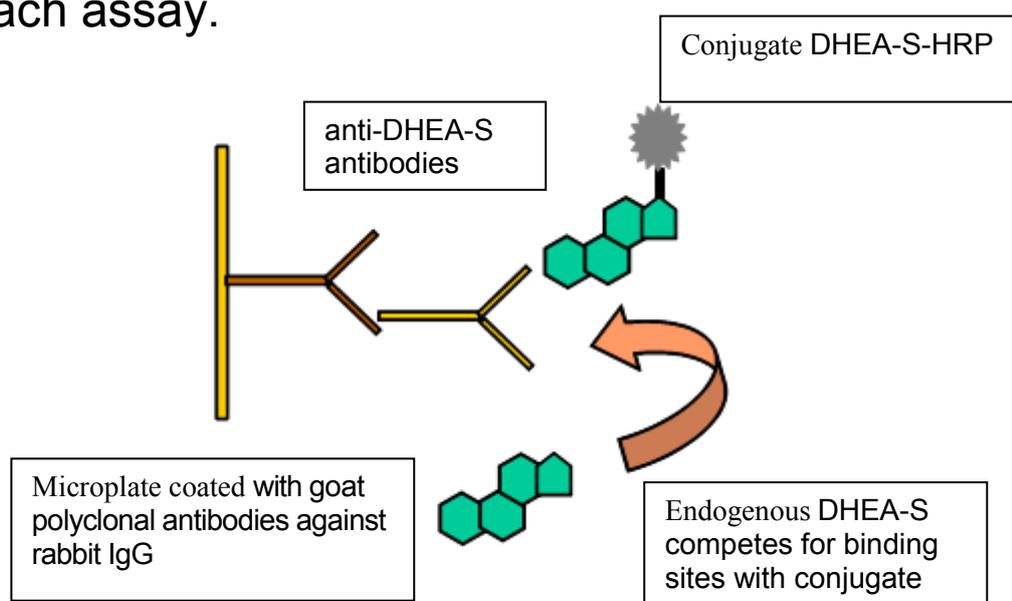
Fig. 1 Assay scheme

4. STORAGE AND STABILITY OF THE KIT

The expiration date of the kit is printed on the box label; the expiration date for each component is printed on the respective label.

The **DHEA-Sulfate kit** should be stored at +2...+8 °C upon receipt, preferably in the original kit box, until the expiration date. Storage at +25 °C is allowed but for no more than 5 days. Shelf life of the kit is 18 months. After initial opening the kit is stable for 12 months if stored at +2...+8 °C.

If used in several separate experiments, after initial opening kit contents should be stored as follows but never used longer than the expiration date:

- unused strips: in a firmly closed resealable zipper bag, concentrated wash solution, stop solution at +2...+8 °C ;
- vials with conjugate, calibrators and control (ready-to-use) and Anti-DHEA-S: at +2...+8 °C for no more than 12 months after opening;
- vials with calibrators and control (reconstituted): at +2...+8 °C for no more than 1 month after opening;
- vial with substrate: at +2...+8 °C until the expiration date, protected from light;
- vials with concentrated Trial, concentrated wash solution and stop solution: at +2...+8 °C until the expiration date;
- wash solution prepared for use: at room temperature (+18...+25 °C) for no more than 5 days or at +2...+8°C for no more than 4 weeks in a firmly closed bottle;
- Trial solution prepared for use: at room temperature (+18...+25 °C) for no more than 5 days in a firmly closed bottle.

Damaged Test Kits

In case of any severe damage of the test kit or components, Astra Biotech GmbH has to be informed in writing, during one week after receiving the kit. Usage of severely damaged single components for a test run is not recommended.

5. SAMPLE COLLECTION AND STORAGE

Collect blood by venipuncture. After clotting, the serum is separated by centrifugation. Plasma, haemolyzed (bright red) or lipaemic (bright red) serum and serum with sodium azide added as preservative should not be used.

Store specimens at +2...+8 °C for no more than 2 days; for longer storage it is recommended to aliquot and freeze them at ≤ -20 °C. Avoid repeated freezing.

6. EXPECTED VALUES

Serum samples collected between 9 and 11 a.m from 200 apparently healthy people (both males and females), were assayed with **DHEA-Sulfate kit**. The results are given in the table below.

Group	Number of patients	Mean (µg/mL)	Range (µg/mL)
<i>Females</i>			
Normally menstruating (19–39 years old)	120	2.4	0.8–3.9
Postmenopausal (49–65 years old)	15	1.35	0.1-2.5
Pregnancy	25	0.7	0.2-1.2
<i>Males (21–39 years old)</i>	40	2.7	1.0-4.2

These limits should be considered as guidelines only.

It is highly recommended for each laboratory to determine its own reference range of DHEA-S concentrations.

7. QUALITY CONTROL

It is recommended to use control samples according to the state and federal regulations. The use of control samples is advised to assure the day to day validity of results.

8. REAGENT PREPARATION

Allow all the reagents to reach room temperature (+18...+25 °C), and then thoroughly stir.

MP Microplate

Keep the **microplate** at room temperature for at least 30 minutes before opening the bag. Place required number of strips onto strip holder. Place unused strips into the resealable zipper bag and reseal duly.

CAL **CONTROL** Calibrators and Control

Liquid calibrators and control are ready to use.

Prepare lyophilized calibrators and control as follows. Gently tap on the vial caps to knock off all the dry matter. Open the vials and carefully place the caps upside down on the clean dry surface. Add 0.5 mL of distilled or deionized water to each vial with lyophilized calibrators and control, close vials with the corresponding caps and leave for 10 min at room temperature without stirring. Then stir gently avoiding foaming, until the dry matter is completely dissolved. Leave for another 10 minutes at room temperature stirring gently periodically. Make sure than no dry matter is left on the caps and walls of the vials.

WASH P Wash solution

Prepare required volume of **wash solution** by dilution of the concentrate 20-fold with distilled or deionized water. For example: 5 mL **WASH P** **20X** + 95 mL water

Mix thoroughly, avoid foaming.

SUB Substrate

Protect **substrate** from direct light.

9. SAMPLE PREPARATION

Allow samples to reach room temperature (+18...+25 °C). Stir samples gently in order to ensure homogeneity.

10. PERFORMANCE CHARACTERISTICS OF THE ASSAY

10.1. Calibration-Traceability

The **DHEA-Sulfate kit** was calibrated against the Working Standard, which had been manufactured by gravimetric method based on weighing purified synthetic DHEA-S into analyte-free matrix.

10.2. Specificity

According to Product specification, provided by the supplier, cross-reaction of anti-DHEA-S antibodies with different steroids is shown in the table to the right.

Steroid	Cross-reaction, %
DHEA-S	100.00
DHEA	50.0
Other related steroids	<1

10.3. Analytical Sensitivity

Analytical sensitivity of **DHEA-Sulfate kit** i.e. concentration that can be distinguished from zero calibrator is 0.04 µg/mL. It was defined as mean OD of 10 replicates of calibrator 0 minus two standard deviations.

10.4. Measurement Range

The **DHEA-Sulfate kits** (manual and Alisei kits) were validated for measurement of DHEA-S concentration within the concentration diapason of 0.04 – 10 µg/mL.

10.5. Measurement Units

In **DHEA-Sulfate kit** the concentrations of calibrators are specified in µg/mL. To convert into µmol/L, multiply the concentration in µg/mL by 2.7.

10.6. Intra- and Inter-Assay Variation (Precision)

For **Intra-Assay CV** determination, 8 serum samples were assayed in 9 replicates each. The results are shown below.

Manual kit

Sample	Mean DHEA-S concentration, $\mu\text{g/mL}$	Intra-Assay CV	
		SD	CV, %
1	1.10	0.080	7.3
2	1.99	0.139	7.0
3	2.55	0.140	5.5
4	4.85	0.189	3.9
5	5.58	0.201	3.6
6	7.83	0.352	4.5
7	9.54	0.410	4.3
8	14.3	0.87	6.1

Alisei kit

Sample	Mean DHEA-S concentration, $\mu\text{g/mL}$	Intra-Assay CV	
		SD	CV, %
1	0.63	0.050	7.9
2	1.03	0.071	6.8
3	1.99	0.127	6.4
4	3.96	0.167	4.2
5	5.14	0.142	2.8
6	7.22	0.249	3.4
7	8.33	0.394	4.7
8	9.47	0.353	3.7

For Inter-Assay CV determination, 8 serum samples were assayed 3 times by different operators with 1-week interval. Each specimen was assayed in 8 replicates. The results are shown below.

Manual kit

Sample	Mean DHEA-S concentration, $\mu\text{g/mL}$			Inter-Assay CV	
	1 assay	2 assay	3 assay	SD	CV, %
1	1.06	1.13	1.25	0.096	8.4
2	2.39	2.12	2.11	0.159	7.2
3	2.75	2.61	2.40	0.173	6.7
4	4.80	4.64	4.40	0.198	4.3
5	5.58	5.80	5.51	0.149	2.6
6	8.40	7.29	8.23	0.596	7.5
7	9.45	9.18	9.67	0.246	2.6
8	15.23	13.89	14.06	0.727	5.0

Alisei kit

Sample	Mean DHEA-S concentration, $\mu\text{g/mL}$			Inter-Assay precision	
	1 assay	2 assay	3 assay	SD	CV, %
1	0.63	0.72	0.68	0.045	6.7
2	1.03	1.20	1.11	0.085	7.6
3	1.99	2.15	1.89	0.131	6.5
4	3.96	3.71	3.78	0.129	3.4
5	5.14	5.00	5.45	0.230	4.4
6	7.22	7.95	7.08	0.467	6.3
7	8.33	8.40	9.19	0.478	5.5
8	9.47	9.06	9.25	0.205	2.2

11. LIMITATION OF THE METHOD

Any clinical diagnosis should not be based on the results of in vitro diagnostic methods alone. For diagnosis establishment, a physician is supposed to consider all available clinical and laboratory findings.

12. SAFETY PRECAUTIONS

This kit is for in vitro diagnostic use only. Operator should follow the manual closely in order to ensure reliable data. The manual is valid for the present kit only, within the listed composition. Any substitution of kit components is not allowed by CE regulations.

- Do not use kit or its components after the expiration date indicated on the label. Take into account stability period for reconstituted reagents.
- Do not mix or use together reagents from different lots of the kit except substrate, stop solution and wash solution.
- Do not use substrate, stop solution and wash solution supplied by other vendors.
- Use only “P”-labeled Astra Biotech wash solution.
- Note that stop reagent is 1 N HCl solution. Avoid contacts with skin and mucosa. In case of contact rinse affected area thoroughly with plenty of water and seek medical advice.
- Take into account the following common procedural notes:
 - always pipette reagents into wells immediately after washing procedure;
 - avoid contamination of the solutions;
 - in case of partial use of the kit, dispense only required volume of the reagent into the tray;
 - do not pour unused reagents back into the original vials;
 - avoid exposure to direct sunlight during incubations;
 - always pipette reagents in the same order to minimize reaction time differences between wells; the total dispensing time for the calibrators, control and samples must not exceed 15 min;

- the incubation temperature for all the immunological reactions must be kept at +37 °C;
- do not touch the bottom of the wells.
- calibrators should be measured in each separate assay. It is also recommended to measure each time DHEA-S concentration in the control.
- remove the adhesive foil carefully to avoid contamination and don't use the adhesive foil repeatedly.
- TMB solution should be colourless. Light colouring of solution is admissible. Avoid direct exposure of substrate to sunlight.
-  Source materials of human origin used for kit components preparation were tested and found negative for HBsAg, anti-HIV and anti-HCV antibodies.
However, none of known laboratory test guarantees absence of these viral agents. Therefore, all kit components and patient's samples should be handled as potentially hazardous.
-  After usage strips, calibrators, control, specimens, and all consumables which contacted with specimens during handling, storage or assay (tubes, vials, gloves, pipette tips etc.) should be collected separately and sterilized by autoclaving. Instead of autoclaving pipette tips may be sterilized by disinfectant treatment. After sterilization all components and expendable materials may be utilized as non-dangerous garbage. Other components of the kit should be discarded into conventional garbage.
-  During manual washing procedure do not discard the contents of the wells directly to drainage. Use a container with disinfectant solution.

-  As the kit contains potentially hazardous material, the following precautions should be taken:
 - do not smoke, eat or drink while performing the assay;
 - always use protective gloves;
 - never pipette material by mouth;
 - in case of spilling, wipe up the spills promptly and wash affected area thoroughly using decontaminant.
- GLP including all general and individual regulations should be applied for the kit usage.

As the kit contains irritant (**CONJ**, **AB**, **CAL**, **CONTROL**), the following precautions should be observed:

- P261 - Avoid breathing spray;
 - P272 - Contaminated work clothing should not be allowed out of the workplace;
 - P280 - Wear protective gloves/protective clothing/eye protection;
 - P363 - Wash contaminated clothing before reuse;
 - P501 - Dispose of contents/container in accordance with national regulation
 - P302+P352 - IF ON SKIN: Wash with plenty of soap and water;
 - P333+P313 - If skin irritation or rash occurs: Get medical advice/attention.
- Precautionary statements according to Regulation EC No 1272/2008.

13. MANUAL TEST (REF 20-01)

13.1 Material Provided

MP	Microplate: 12 breakable 8-well strips (total 96 wells) coated with goat polyclonal antibodies against rabbit IgG	1 pcs
CONJ	Conjugate: solution containing DHEA-S conjugated with HRP	14 mL, ready to use
0-5 CAL	DHEA-S calibrators: protein-based solution or lyophilized preparations containing known DHEA-S concentrations - 0; 0.1; 0.5; 1; 2.5; 10 µg/mL (approximate values – do not use for evaluation of real assay data). For lot specific DHEA-S concentrations see values in the Quality Control Sheet	6 vials, 0.5 mL each; ready to use or lyophilized
CONTROL	DHEA-S control: protein-based solution or lyophilized preparation containing known DHEA-S concentration. For lot specific ranges of DHEA-S concentrations see values in the Quality Control Sheet	0.5 mL, ready to use or lyophilized
SUB	Substrate (TMB solution): 3,3',5,5'-tetramethylbenzidine solution in citrate buffer containing hydrogen peroxide	14 mL, ready to use
WASH P 20X	Wash solution P, 20X concentrated: surfactant in buffered saline, surfactant in buffered saline, sufficient for preparation of 560 mL of solution	2x14 mL, concentrated
STOP	Stop solution: 1 N HCl solution	14 mL, ready to use
AB	Solution of anti-DHEA-S antibodies. Contains rabbit antiserum	14 mL, ready to use
	Adhesive foil	2x1 pcs, (optional)

13.2 Equipment and Materials Required but not provided

- 1-channel calibrated variable precision pipettes, with disposable tips;
- 8-channel calibrated variable precision pipette, with disposable tips;
- microplate incubator (+37 °C) or microplate incubator-shaker (+37 °C, 500–800 rpm);
- manual or automatic equipment for rinsing wells;
- calibrated microplate reader (450 nm);
- vortex tube mixer;
- deionized or distilled water;
- graduated beaker and cylinder of appropriate volume;
- latex or plastic gloves;
- trays for pipetting reagents with 8-channel pipette;
- disinfectant;
- absorbent material (for manual wash).

13.3 Test Procedure

The **DHEA-Sulfate kit** is designed for 96 tests. This is sufficient for 40 unknowns, 6 calibrators, 1 control and 1 blank (control of OD of TMB solution) in duplicates, provided that all the strips are used simultaneously.

13.3.1 Assay Procedure

Assay Protocol 1

(see also Assay scheme to Assay Protocol 1, section 13.5)

All samples should be tested in duplicates.

- A.** Pipette **50 µL** of **calibrators** **CAL** (0-5), **control** **CONTROL** and prepared patient's samples in duplicates into the respective wells; **Leave wells A1-A2 empty for blank!**
- B.** Pipette **100 µL** **conjugate** **CONJ** into each well, except wells A1-A2.
- C.** Pipette **100 µL** **anti-DHEA-S** **AB** into each well, except wells A1-A2.

Note: *total time of dispensing must not exceed 15 minutes, otherwise the test result may be unreliable, because the time of incubation will substantially vary for different samples.*

Note: *If an adhesive foil is used, put it now onto the plate to seal the cavities.*

- D.** Incubate strips for 60 minutes while shaking (500–800 rpm) at +37 °C.

Note: *If an adhesive foil is used, remove it now from the plate.*

- E.** Wash 4 times, as described below.
- F.** Pipette **100 µL** **substrate** **SUB** into each well; incubate strips **at room temperature** (+18... +25 °C) **in the dark for 15-30 minutes**, depending on the colour intensity, or **10 minutes while shaking (500-800 rpm) at +37 °C**.
- G.** Pipette **100 µL** **stop solution** **STOP** into each well (including blank) in the same sequence and at the same speed as

used for dispensing TMB substrate. **Shake for 1–2 min at room temperature.**

H. Read OD at 450 nm within 20 min.

Assay Protocol 2: (without shaking)

(see also Assay scheme to Assay Protocol 2, section 13.6)

All samples should be tested in duplicates.

A. Pipette **50 µL** of **CAL** (0-5), **CONTROL** and patient's samples in duplicates into the respective wells; **Leave wells A1-A2 empty for blank!**

B. Pipette **100 µL** **CONJ** into each well, except wells A1-A2.

C. Pipette **100 µL** **AB** into each well, except wells A1-A2.

Note: *total time of dispensing must not exceed 15 minutes, otherwise the test result may be unreliable, because the time of incubation will substantially vary for different samples.*

Note: *If an adhesive foil is used, put it now onto the plate to seal the cavities.*

D. Incubate strips for **60 minutes at +37 °C** (pre-shake for 1-2 minutes at room temperature).

Note: *If an adhesive foil is used, remove it now from the plate.*

E. Wash 4 times, as described in the Instructions for use for DHEA-Sulfate kit (see 13.3.2. Wash Procedure).

F. Pipette **100 µL** **SUB** into each well; incubate strips **at room temperature** (+18... +25 °C) **in the dark for 15-30 minutes** depending on the colour intensity.

- G.** Pipette **100 μ L** **STOP** into each well (including blank) in the same sequence and at the same speed as used for dispensing TMB substrate. **Shake for 1–2 min at room temperature.**
- H.** Read **OD** at **450 nm** within **20 min.**

13.3.2 Wash procedure

It is advisable to use an automatic microplate washer set at 4 wash cycles and a volume of 300 μ L of wash solution per well per cycle. If an automatic washer is not available, the wash procedure can be carried out manually as follows:

- remove the contents of the wells into a container with disinfectant;
- dispense 300 μ L of wash solution, (section 8), into each well, shake the plate carefully for 5–10 sec and remove the contents of the wells; repeat 4 times;
- strike the wells sharply on absorbent material to remove any liquid residue.

13.4 Data Processing

If the reader cannot be adjusted to zero using the substrate blank in wells A1-A2, subtract mean OD value of wells A1-A2 from all OD values before further calculations.

Example:

OD (Cal 0) measured = 2.28 and OD (blank) = 0.06;

OD (Cal 0) calculated = $2.28 - 0.06 = \underline{2.22}$

13.4.1 Data Reliability (for OD measured at 450 nm)

The data should meet the following criteria:

- average blank OD (in wells A1-A2) ≤ 0.100 ;
- average OD of Cal 0 ≥ 1.0 (after blank subtraction);
- control's concentration must fall within the acceptability ranges that is shown on the Quality Control Sheet.

If the data obtained do not meet the criteria, the results are considered unreliable and the test should be repeated.

13.4.2 Quantitative Determination

Specialized software for quantitative determination is recommended. Mean OD values of the calibrators at 450 nm versus their respective DHEA-S concentrations using **4PL or 5PL fit** (Fig. 2 Typical standard curve).

Any extrapolation of the standard curve to DHEA-S above the nominal value of the Calibrator 5 (approximately 10 $\mu\text{g/mL}$) is forbidden.

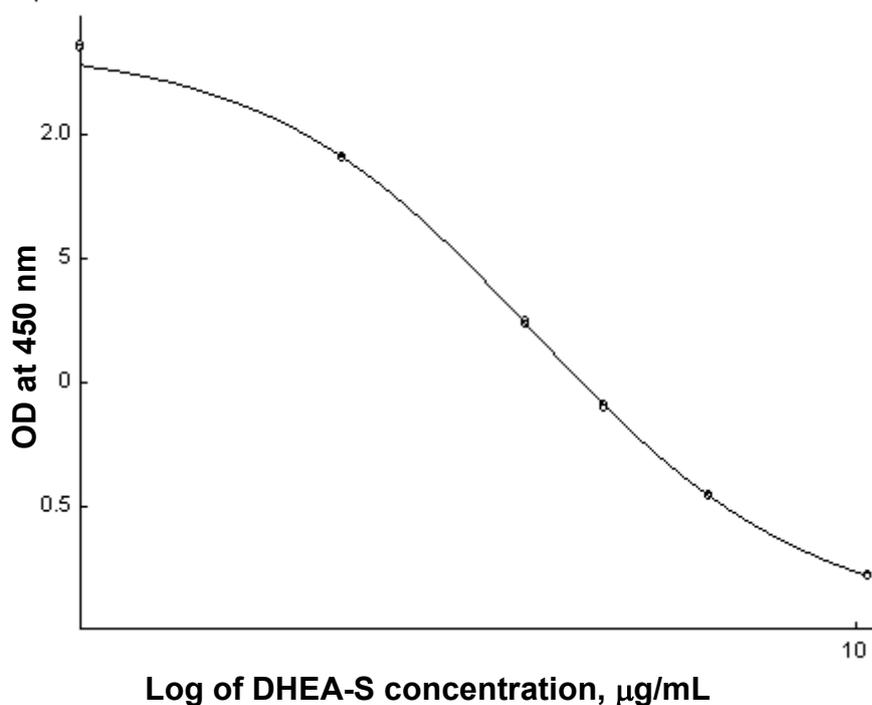
Fig. 2 Example of typical standard curve.
Do not use for evaluation of real assay data!

13.5 Assay scheme to Assay Protocol 1

		Wells		
		«Blank»	CAL CONTROL	Samples
Reagents				
CAL		–	50 µL	–
CONTROL		–	–	–
Samples		–	–	50 µL
CONJ		–	100 µL	100 µL
AB		–	100 µL	100 µL
Incubation No.1	60 min, +37 °C, 500–800 rpm			

WASH P (diluted)	4 x 300 μ L		
SUB	100 μ L	100 μ L	100 μ L
Incubation No.2	15- 30 min, +18...+25 °C, in the dark		
	10 min, +37 °C, 500–800 rpm		
STOP	100 μ L	100 μ L	100 μ L
Stirring	1–2 min, +18...+25 °C		
OD measuring	450 nm		
Calculations	Corresponding software		

13.6 Assay scheme to Assay Protocol 2 (without shaking)

Wells	«Blank»	CAL CONTROL	Samples
Reagents			
CAL	–	50 μ L	–
CONTROL	–	–	–
Samples	–	–	50 μ L
CONJ	–	100 μ L	100 μ L
AB	–	100 μ L	100 μ L
Incubation No.1	60 min, +37 °C (pre-shake for 1-2 minutes at room temperature)		
WASH P (diluted)	4 x 300 μ L		
SUB	100 μ L	100 μ L	100 μ L
Incubation No.2	15–30 min, +18...+25 °C, in the dark		

STOP	100 μ L	100 μ L	100 μ L
Stirring	1–2 min, +18...+25 °C		
OD measuring	450 nm		
Calculations	Corresponding software		

14. AUTOMATIC TEST (REF 20-01A)

14.1. Material Provided

MP	Microplate: 12 breakable 8-well strips (total 96 wells) coated with goat polyclonal antibodies against rabbit IgG	1 pcs
CONJ	Conjugate: solution containing DHEA-S conjugated with HRP	14 mL, ready to use
0-5 CAL	DHEA-S calibrators: protein-based solution or lyophilized preparations containing known DHEA-S concentrations - 0; 0.1; 0.5; 1; 2.5; 10 µg/mL (approximate values - do not use for evaluation of real assay data). For lot specific DHEA-S concentrations see values provided in the Quality Control Sheet.	8 vials, CAL 2,3- 2×0.5mL; CAL 0,1,4,5- 1× 0.5 mL; ready to use or lyophilized
CONTROL	DHEA-S control: protein-based solution or lyophilized preparation containing known DHEA-S concentration For lot specific ranges of Prolactin concentrations see Quality Control Sheet	2 × 0.5 mL, ready to use or lyophilized
WASH P 20X	Wash solution P, 20X concentrated: surfactant in buffered saline, surfactant in buffered saline, sufficient for preparation of 1000 mL of solution	50 mL, concentrated
SUB	Substrate (TMB solution): 3,3',5,5'-tetramethylbenzidine solution in citrate buffer containing hydrogen peroxide	14 mL, ready to use
STOP	Stop solution: 1 N HCl solution	19 mL, ready to use
TRIAL 5000X	Trial solution, 5000X concentrated: Solution of detergent	It is delivered by separate order

AB	Solution of anti-DHEA-S antibodies. Contains rabbit antiserum	14 mL, ready to use
-----------	---	------------------------

Note: *Extra vials of **CAL** 2, 3 and **CONTROL** are provided for recalibration on reference calibration curve. For detailed information see instructions for use for analyser “Alisei”.*

14.2. Equipment and Materials required but not provided

- Automatic analyser for ELISA kits on microplate (analyser “Alisei”);
- multipurpose polypropylene tubes 12x75 volume 5.5 mL;
- **TRIAL 5000X** Trial solution, 5000X concentrated*;
- deionized or distilled water;
- latex or plastic gloves.

14.3. Test Procedure

The **DHEA-Sulfate kit** is designed for 96 tests. This is sufficient for 88 unknowns, 6 calibrators, 1 control and 1 blank (OD of TMB solution) in monoplicates, provided that all the strips are used simultaneously.

14.3.1. Reagent Preparation

Allow all the reagents to reach room temperature (+18...+25 °C), and then thoroughly stir.

Note: *Preparation of other reagents see in section 8.*

TRIAL Working solution of Trial for washing internal hydraulic lines and needles of automatic analyser “Alisei”.

Prepare required volume of **Trial solution** by dilution of the concentrate 5000-fold with distilled or deionized water before analysis:

* Reagent is not included in the kit, it is delivered by separate order.

2 mL **TRIAL 5000X** + 9998 mL water

Mix thoroughly, avoid foaming.

14.3.2. Assay Procedure for Automatic Test

While using for the procedure analyzer “Alisei”, refer to its relative manual.

The analysis on ELISA analyzer “Alisei” is completely automatic: pipetting of reagents, washing, incubation, OD measurement and analysis of results. The program of calculation DHEA-S concentration is hardwired in analyzer memory.

14.4. Data Processing

14.4.1. Data Reliability (OD measured at 450 nm)

See criteria in section **13.4.1**.

14.5. Safety precautions

- If kit for analyser “Alisei” is used in several separate experiments, it is necessary to take reagents from analyser “Alisei” immediately after pipetting them in the wells of all plates because liquid evaporates from vials. Put the reagents in refrigerator;
- Recalibration using calibration curve, obtained with a kit of any other lot, is not permitted.

August 26, 2019

Astra Biotech GmbH
Rudower Chaussee 29,
12489 Berlin, Germany
Telefon: +49 (0)30 74696509
E-Mail: info@astrabiotech.de
www.astrabiotech.de